PRODUCTINFORMATIE EPI BASIC

Dutch

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PRODUCT INFORMATION EPI BASIC

English

Description: The Epi Basic brace, also called the tennis arm brace, is an elastic brace made of high-quality, elastic materials. The brace contains a pad, which exerts pressure on the muscle abdomen of the forearm. This reduces the pressure on the tendon attachment and reduces pain. The Epi Basic is easy to apply by means of a Velcro system.

Sizing:

SIZES	ART.NR.
ONZE SIZE	PO 31304

Indications: Epicondylalgia lateralis (formerly epicondylitis lateralis, also called tennis elbow), Epicondylalgia medialis (golfer's elbow)

Manual:

1. Open the Velcro closure and slide the arm through the loop.
2. Place the pad with the opening towards the elbow joint, about 5 centimeters below the joint gap, on the muscle abdomen.
3. Then close the Velcro to create compression. Make sure that the brace is not too tight. Too much compression can cause reduced blood flow.

Important information and warnings:

This medical product is intended for use by one individual patient. The product should be used in accordance with the advice and instructions of your physician. The product should be installed by a trained professional the first time.

In cases where the product is recommended by the doctor to be used at night, it must be ensured that it does not interfere with blood circulation. The product should be detached in case of numbness and removed if necessary. Consult your doctor in case of persistent complaints.

The product should not be exposed to excessive heat or flame. Contact with oily or acidic substances or lotions containing pomades should be avoided.

This product must not be used outside its scope or combined with any other product. It should only be used according to the original indications.

Daily use and duration of application should be determined by the physician. Excessive and prolonged use can cause muscle weakness. Always consult your doctor for the correct wearing schedule.

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