## PRODUCTINFORMATIE MALLET FINGER VINGERSPAK

Dutch

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**Indicaties:** Mallet finger, hamervinger

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## PRODUCT INFORMTION MALLET FINGER SPLINT

English

**Description:** The Mallet finger splint is suitable for Mallet fFnger (mallet finger) and other complaints where the finger must be supported as much as possible. The injured finger in question receives maximum support. The rest of the fingers are also kept completely free, giving the finger the maximum rest it needs. The adjustable strap keeps the finger splint in place.

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**Indications:** Mallet finger, hammer finger

### Manual:

1. Measure the circumference of the finger before wearing this finger splint

### Important information and warnings:

This medical product is intended for use by one individual patient. The product should be used in accordance with the advice and instructions of your physician. The product should be installed by a trained professional the first time.

In cases where the product is recommended by the doctor to be used at night, it must be ensured that it does not interfere with blood circulation. The product should be detached in case of numbness and removed if necessary. Consult your doctor in case of persistent complaints.

The product should not be exposed to excessive heat or flame. Contact with oily or acidic substances or lotions containing pomades should be avoided.

This product must not be used outside its scope or combined with any other product. It should only be used according to the original indications.

Daily use and duration of application should be determined by the physician. Excessive and prolonged use can cause muscle weakness. Always consult your doctor for the correct wearing schedule.

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