PRODUCTINFORMATIE PATELLA BASIC

Dutch

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PRODUCT INFORMATION PATELLA BASIC

English

Description: Patella Basic is a patella strap for moderate grip and stability made of durable and light neoprene. Provides extra fixation on the knee and prevents constrictions in the blood circulation. Special padding supports the patellar tendon and relieves pain through compression.

Sizing:

SIZE	ART.NR.
ONE SIZE	PO 31512

Indications: Osgood-Schlatter, Jumpers knee, inflammation of the patellar tendon (tendinitis), patellar tendon overload, support in sports or daily activities

Manual:

1. Open the Velcro connections
2. Place the U-shaped pad on the tendon, below the kneecap with the opening facing up.
3. Then put the band around the knee and close the Velcro fastener. Make sure that the compression is comfortable. Make sure the clamp is not too tight. Excessive compression can lead to decreased blood flow.

Important information and warnings:

This medical product is intended for use by one individual patient. The product should be used in accordance with the advice and instructions of your doctor. The product should be installed by a trained

professional the first time.

In cases where the product is recommended by the doctor to be used at night, it must be ensured that it does not interfere with blood circulation. The product should be detached in case of numbness and removed if necessary. Consult your doctor in case of persistent complaints.

The product should not be exposed to excessive heat or flame. Contact with oily or acidic substances or lotions containing pomades should be avoided.

This product must not be used outside its scope or combined with any other product. It should only be used according to the original indications.

Daily use and duration of application should be determined by the physician. Excessive and prolonged use can cause muscle weakness. Always consult your doctor for the correct wearing schedule.

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