PRODUCTINFORMATIE RIB BAND

Dutch

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PRODUCT INFORMATION RIB BAND

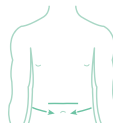
English

Description: The Rib Band is a high quality rib bandage made of elastic fabric. Easy to attach and remove thanks to the practical Velcro system. The rib band supports the abdominal muscle tissue and weakened. Supports the abdominal wall to heal after surgery.

Sizing:

SIZE	BELLY SIZE	ART.NR.
S	75 - 85 cm	PO 31420-D S
M	85 - 95 cm	PO 31420-D M
L	95 - 105 cm	PO 31420-D L
XL	105-115 cm	PO 31420-D XL
XXL	115 - 125 cm	PO 31420-D XXL

Indications: Rib contusion, Rib fracture, Pain in muscles (chest)

Manual:

1. Grab both ends and fasten the bandage from behind the back, towards the abdomen with the Velcro fastener.
2. Make sure that there is sufficient tension on the bandage, that support is given and at the same time there is no hindrance when breathing.

Important information and warnings:

This medical product is intended for use by one individual patient. The product should be used in accordance with your doctor's advice and instructions. The product should be installed by a trained professional the first time.

In cases where the product is recommended by the doctor to be used at night, it must be ensured that it does not interfere with blood circulation. The product should be detached in case of numbness and removed if necessary. Consult your doctor in case of persistent complaints.

The product should not be exposed to excessive heat or flame. Contact with oily or acidic substances or lotions containing pomades should be avoided.

This product must not be used outside its scope or combined with any other product. It should only be used according to the original indications.

Daily use and duration of application should be determined by the physician. Excessive and prolonged use can cause muscle weakness. Always consult your doctor for the correct wearing schedule.

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